

DenTek Oral Care Products

Win a DenTek Bundle >>

Comfort Clean Floss Picks

Gentle on teeth and gums, tough on food and plaque.

DenTek Comfort Clean Floss Pick's silky floss tape is gentle on teeth and gums while comfortably removing food and plaque.

Flossing is important to the health of your teeth and gums. DenTek's Comfort Clean Floss Picks make it comfortable and easy:

- Ultra thin Silky Comfort Floss Tape that slides smoothly between teeth
 - Minty flavour that leaves your mouth fresh
 - Textured Pick that cleans between teeth and helps stimulate gums
 - Tongue Cleaner that helps fight bad breath



Instructions for Use

Flossing Instructions:

- Gently slide floss up and down between teeth allowing the floss to wrap around teeth.
 - Clean above and below the gum line.
 - Rinse as necessary.

Pick Instructions:

- Place pick between teeth and gently press against the gum.
- Use an in and out motion to remove food particles and stimulate gums.
- Rinse as necessary.

Tongue Cleaning Instructions:

- Extend your tongue.
- Place the Tongue Scraper on the back of your tongue and pull forward, skimming the surface of your tongue.
- Rinse as necessary.

© 2014 DenTek Oral Care, Inc. All Rights Reserved.

v>