

DenTek Oral Care Products

Win a DenTek Bundle >>

Complete Clean For Back Teeth Floss Picks

Conquer tight spaces even between the hardest to reach back teeth.

DenTek Complete Clean for Back Teeth Floss Picks combine ultra strong floss with a Y shaped handle to help reach and clean between back teeth.

Don't neglect your back teeth because they are hard to reach. DenTek Complete Clean Back Teeth Floss Picks are the solution for tight spaces anywhere in the mouth:

- Curved Y shaped handle to help reach back teeth
- Break Resistant Textured Floss
- Fluoride coating
- Bristled Pick brushes deep between teeth
- Minty flavour that helps leave your mouth feeling fresh



Instructions for Use

Flossing Instructions:

- Gently slide floss up and down between teeth allowing the floss to wrap around teeth.
 - Clean above and below the gum line.
 - Rinse as necessary.

Pick Instructions:

- Place pick between teeth and gently press against the gum.
 - Use an in and out motion to remove food particles and stimulate gums.
 - Rinse as necessary.

© 2014 DenTek Oral Care, Inc. All Rights Reserved.

v>