

DenTek Oral Care Products

Win a DenTek Bundle >>

Complete Dental Repair Kit

Thin Set Included

The Dental First Aid Kit contains various items to temporarily replace a lost filling, cap, crown or inlay until you have a chance to visit your dentist.

The kit contains an applicator, Thin Set Cap & Crown Cement, Temparin Max Lost Filling & Loose Cap Repair, Eugenol, and a Tooth Saver vial for a dislodged tooth.

This product is recommended for:

- Lost fillings
- Loose caps and crowns
- Dislodged tooth
- Temporary use



Instructions for Use

Caution: To avoid infection and/or serious injury, this product should not be used if there is throbbing pain, ulceration or swelling in the affected area. If any of these symptoms exist, consult your dentist immediately. This product should not be placed on open wounds. Do not swallow. Do not use if allergic to Eugenol. READ ALL OF THE DIRECTIONS CAREFULLY BEFORE USE. Wash and dry your hands. Keep out of reach of children. Do not get into eyes.

Tooth Saver: If tooth is dislodged, collect the tooth and do not try to clean it, there could be important soft tissue that will help in the healing process. Fill Tooth Saver container with milk or your own saliva. Place broken or whole tooth in container making certain that milk or saliva is completely covering tooth. Contact your dentist immediately. Chances of successful replantation are best if you see the dentist within 1 hour of incident.

Temparin Max : This is a temporary remedy meant to replace a lost filling, cap, crown or inlay until you have a chance to visit your dentist and should only be removed by a dental professional. Use only what is needed to repair the dental emergency, and save the rest for a future application. Visit dentist within 48 hours of use. If irritation persists, inflammation develops, or if fever and infection develop stop use and ask a dentist or doctor.

Directions to replace a lost filling: Wash the cavity area with warm water to remove food particles. Saturate tip of a fresh cotton swab with liquid from the Eugenol vial and apply to tooth. Avoid touching surrounding gum tissue, cheek, or tongue with wet cotton swab. Do not leave cotton swab in mouth. The Eugenol will help the filling material set up and adhere to the tooth enamel. Cut with applicator or pinch a small amount of Temparin Max from the vial, and roll into a ball between fingers. Slightly overfill the cavity with Temparin Max and tamp into place with applicator tip. Close your teeth together, as if you were biting down on a piece of paper, and gently chew or grind on the replaced filling a couple times to create a comfortable bite. Use a moist cotton swab to remove excess Temparin Max from around the area. Moisture will accelerate the set.

Temparin Max will set softly in your mouth within a few minutes, but avoid chewing on the repair for 2-3 hours to allow the material to fully set. Clean applicator with a DRY paper towel and save for future use.

Directions to secure loose crowns, caps, or inlays: Try the crown, cap, or inlay first to see if you can fit it accurately back onto the tooth surface. If you are not able to do so, or if remaining tooth is cracked, then do not proceed further. If the crown has a post attached to it or if there is tooth surface remaining inside the crown, Temparin Max may not adhere. Carefully rinse the tooth area and the inside of the crown, cap, or inlay with warm water, and do not dry. Cut with applicator or pinch a small amount of Temparin Max from the vial, roll into a ball between fingers, and place evenly inside the crown, cap, or inlay. Place the crown, cap, or inlay on the tooth and bite down, applying pressure to push into position. Temparin Max will begin to harden within a few minutes, but avoid chewing on the repair for at least 2-3 hours to allow the material to set. Use a toothbrush to gently clear any excess material from around the crown, cap, or inlay. Clean applicator with a DRY paper towel and save for future use.

Thin Set Directions: 1. Test fit the dental restoration securely on the tooth surface. Without a secure fit, DO NOT PROCEED. 2. Rinse the tooth area and inside of the restoration area with warm water. 3. Dry area with a cotton swab. 4. Use the applicator to mix all of Part A and Part B together, on a small paper plate. Mix until a smooth consistency is achieved. NOTE: AT THIS POINT YOU HAVE 1 MINUTE TO PLACE DENTAL WORK IN YOUR MOUTH BEFORE CEMENT BEGINS TO HARDEN. 5. Use the applicator to place Thin Set evenly inside the dental restoration. 6. Place the restoration on clean, dry tooth and gently bite down. 7. Hold dental work in place for 3 minutes for Thin Set to set. 8. Use a toothbrush to gently clear any excess material from the restoration once the cement begins to harden. 9. Use dental floss 30 minutes later to clean the restoration. 10. Avoid chewing on repair for 1 hour to allow material to set. Thin Set will begin to harden but will not completely set for 1 hour. 11. Clean applicator with a DRY paper towel and save for future use.

Ingredients for Temparin Max: Zinc Oxide Powder, Calcium Sulfate, Petroleum Jelly, Potassium alum, Paraffin, Aluminum

Sulphate, Aluminum Phosphate, Menthol Crystals, Eugenol, Yellow Iron Oxide

Ingredients for Eugenol: Eugenol, Sesame Oil

© 2014 DenTek Oral Care, Inc. All Rights Reserved.

v>