

# DenTek Oral Care Products

**Win a DenTek Bundle** >>

## Thin Set Cap & Crown Cement

### Repair Loose Caps and Crowns

Thin Set Cap & Crown Cement is a temporary remedy meant to replace a lost cap or crown until you have a chance to visit your dentist.

This product is recommended for:

- Loose caps and crowns
- Temporary use



## Instructions for Use

Use: Temporarily secures caps, crowns, laminates. Seek professional dental advise within 48 hours.

Caution: Do not use if there is infection, throbbing pain, ulceration, or swelling and consult a dentist immediately. This product should not be placed on open wounds. **READ ALL OF THE DIRECTIONS CAREFULLY BEFORE USE!** Wash and dry hands. Use only what is needed to repair the dental emergency. **KEEP OUT OF**

REACH OF CHILDREN.

Directions:

- Test fit the dental restoration securely on the tooth surface. Without a secure fit, DO NOT PROCEED.
- Rinse the tooth area and inside of the restoration area with warm water.
- Dry area with the cotton swab.
- Use wooden applicator stick to mix equal portion of content of Part A and Part B onto plastic part of package. Mix until smooth consistency is achieved. NOTE: AT THIS POINT YOU HAVE 1 MINUTE TO PLACE DENTAL WORK IN YOUR MOUTH BEFORE CEMENT BEGINS TO HARDEN.
- Place Thin Set evenly inside the dental restoration with applicator.
- Place the restoration on clean, dry tooth and gently bite down.
- Hold dental work in place for 2 minutes for Thin Set to set.
- Use a toothbrush to gently clear any excess material from the restoration once the cement begins to harden.
- Use dental floss 30 minutes later to clean the restoration.
- Avoid chewing for 1 hour to allow material to set. Thin Set will begin to harden but will not completely set for 1 hour.

© 2014 DenTek Oral Care, Inc. All Rights Reserved.

v>